

Microscopic Single Frosted Slides

PRODUCT INFORMATION:

PS7107-72 slides per pack

INTENDED USE

The Frosted Single Slides are high-quality pre-cleaned glass slides which are frosted at one end on one side only which facilitates marking specimens to help ensure specimen identification and traceability. The Frosted single microscopic slides are all ready to use which may be used across a wide range of applications which include Pathology, Cytology, Microbiology and Hematology laboratories which accommodates various marking methods such as conventional labeling, writing, or printing. With the one end single-sided frosting, the microscopic slides are thoughtfully tailored to suit the laboratory needs.

SUMMARY AND EXPLANATION

The Frosted single slides have one end surface frosted only on one side with ground edges, which can be used as a label field. The frosted tip of the slides allows the user to easily identify the specimen by writing directly on the slide. These are pre-cleaned clear glass slides with grounded edge of 25.4 x 76.2mm (1" x 3") and (1mm - 1.2mm) thickness which help maintain specimen integrity and achieve accurate results.

STORAGE AND HANDLING

The Frosted single slides should be always stored in a dry place, to avoid sticking and corrosion of the glass surface. The changes in temperature and humidity results in moisture contamination of the slides. Slides should not be stored on concrete floors and should be kept as far away as possible from heat and air conditioning to avoid large variations in temperature and humidity. Do not store slides close to solvents as this might turn their surfaces hydrophobic.

RECOMMENDED PROTOCOL

- Slides are normally ready for use.
- Labelling fields on the slides serve to clearly identify the preparations, so they must be labelled appropriately.
- The slides can be used according to the respective laboratory validated procedures.
- Avoid mishandling of slides to minimize scratches and breakage.
- PathnSitu Frosted single slides are developed to work with pathology, cytology, microbiology and hematology laboratories of any source.

PRECAUTIONS

- The slides are intended for single use by qualified personnel only.
- They must not be used more than once and, after use they must be properly stored/disposed.
- Wear gloves to reduce the risk of injury from sharp glass edges as much as possible
- Any broken slides must be discarded into containers which are purpose-built for sharps.

TROUBLESHOOTING

Follow the reagent specific protocol recommendations according to data sheet provided. If unusual results occur, contact PathnSitu Technical Support at 040-2701 5544 or techsupport@pathnsitu.com

LIMITATIONS AND WARRANTY

There are no warranties, expressed or implied, which extend beyond this description. PathnSitu is not liable for property damage, personal injury, or economic loss caused by this product.

EXPLANATION OF SYMBOLS

CAT# - Catalogue number

Qty: Quantity of box/ No. of slides per box